

nenoTM

*inspired by Children
designed by Parents*

bene

NENO BENE PORTABLE MESH
NEBULIZER / INHALER



*for newborns
and for the whole family*



*battery case
for cordless use*



*medical
device*



INNOVATING PRODUCT
IN THE NEBULIZER CATEGORY

Welcome, Neno Bene !

*Rekomendasi
Maja Hyzy*

Maja Hyzy

Brand ambassador Neno 2021



neno®

FROM THE FIRST DAYS OF LIFE

Neno Bene is another medical device of our brand. The newest mesh nebulizer is an innovative product on the European market, as it was designed not only for treating adults, but also for inhaling therapy for children. It is a medically suitable for the newborns' needs from their 1st month of life.

ASTHMA, ALLERGY, UPPER RESPIRATORY PROBLEMS

Asthma: approximately 10 % of population has a diagnosed asthma in early or developed stage. Nebulizers help asthma patients to cope with the symptoms, by inhaling the vapours of saline solutions or steroids.

Allergies: allergy to fur, mites, pollen – almost every person knows somebody with such a problem. Sneezing, coughing, problems with breathing, runny nose are typical troubles for those suffering from allergy. Neno Bene enables fast and easy medication administering, at the same time unblocking the respiratory system, and helping to remove phlegm or mucus.


Pharyngitis: throat problems often result in pain and difficulty to swallow. Neno Bene eliminates this problem – the medication is administered with a mist directly to the respiratory system.





POST-COVID TREATMENT

The pandemics have changed our approach to health and to taking care of the respiratory system. Using the mesh nebulizer not only enables an easier and more steady breathing, but also accelerates the healing of the respiratory system of post-Covid patients, and may be used as an additional form of therapy in the lower respiratory tract illnesses.

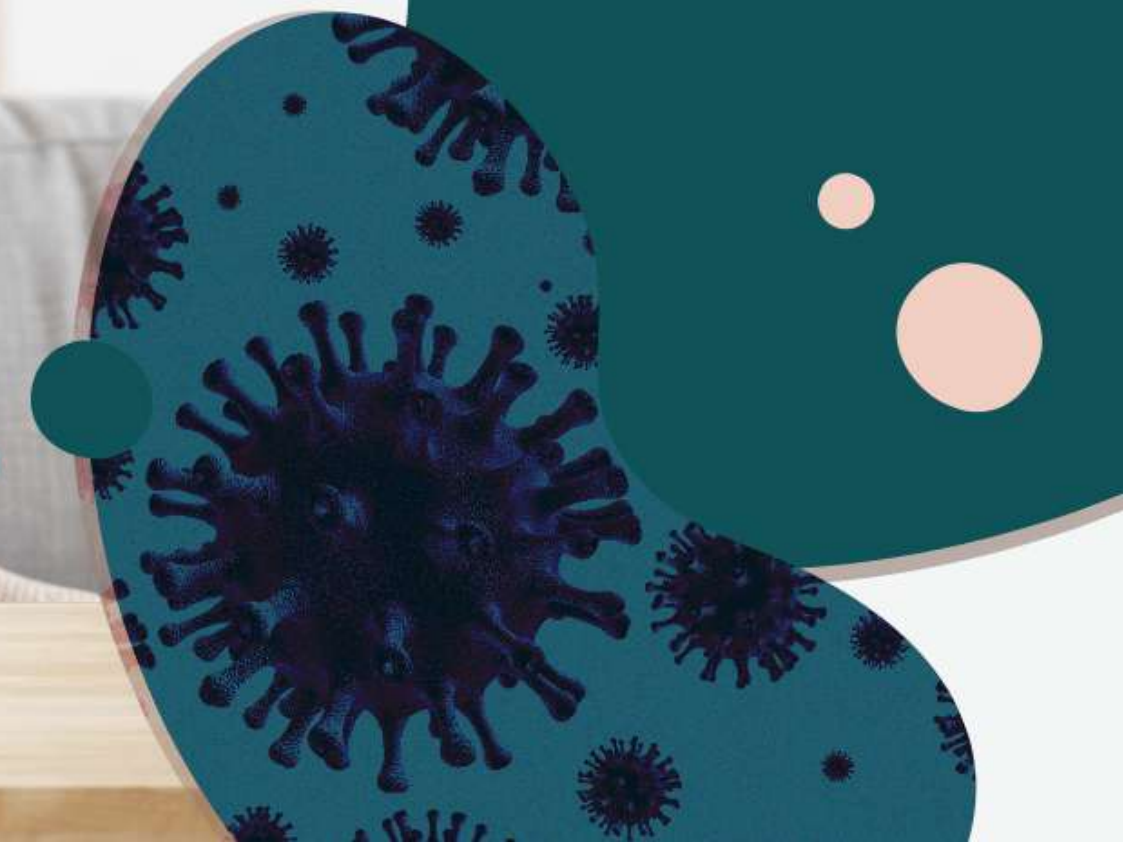



NEBULIZING AT HOME

Nebulizing is a form of inhalatory treatment, which is pain-free and non-invasive. It shortens the time needed for the medications to be absorbed by the patient's system, by creating the mist which is inhaled, and eventually transported with the oxygen into the blood. It's more efficient than the traditional methods of medication administering. Such a procedure can be easily carried out at home. You just need to plug it in, put a mask on, and „voila” - just breathe...



PORTABLE SOLUTION

Neno Bene mesh nebulizer is a small, lightweight, and fully portable – therefore it is a perfect solution not only for home use, but also for travelling or using while you're out. You can just plug it to a powerbank or to a battery case, and use it portably. The whole device will fit in a pocket or a purse, but Neno Bene also has a textile bag for storing the device and all masks.





ALWAYS AVAILABLE - ALSO DURING TRAVEL

Neno Bene mesh nebulizer is a very small device – you can easily keep it at home, and also take it with you when travelling. It can be charged from batteries, you can plug it into the electric socket, or charge it via' an USB-C port from a laptop, or from a car charger. Neno Bene is a good solution.



THERAPY DURING WORK

Neno Bene nebulizer is a device for frequent use. It consists of the small medication container, silicone mask, and an USB power cord which can be plugged to a laptop or a powerbank. Thanks fo that you can have it always with you while at work, and use it whenever needed. You just need to breathe freely. Simple setup enables you to have your hands unoccupied, thus enabling you to keep working.



FOR THE WHOLE FAMILY

Nebulizing is a perfect way for moisturizing the respiratory tract and cleaning it during infections. Another advantage of nebulizing is the lack of stomach irritating risk, so it can be used both by youngest children unable to swallow a pill, and by adults with sensitive digestive system. Nebulizing with Neno Bene can also be carried out for passive patients. The set contains 3 sizes of masks – for adults, kids, and newborns.



WELL FITTING MASK

Nebulizing with Neno Bene is done with a tiny mask, and not with a teat, which helps to achieve a more efficient and comfortable use – both for the parents and for the baby. Newborns may not breathe through the nose, so nebulizing with a teat may be less effective than nebulizing with a mask.







INNOVATIVE MEDICATION CONTAINER

Neno Bene is a mesh nebulizer – it uses a mesh to create a mist, by splitting the fluid containing medication into a mist of tiny particles (1 - 5 μ m). The innovative shape of the container secures a higher efficiency and maximises the use of medication. This part should be replaced after 30 hours of work – to ensure the optimal efficiency of the device. 3 masks are made of elastic material, which adapts to the individual shape of user's face. The mask has holes securing proper breathing. The masks can be easily kept clean by washing it in warm (NOT hot) water with soft detergent, and storing it in a bag after getting it dry.



FOR ALL GENERATIONS

The mesh nebulizer can be used by anybody. The set contains 3 masks of different sizes – for adults, kids, and for newborns. Neno Bene can be used in a comfortable body position, avoiding the traditional bending over a steaming cup. Nebulizing is breathing the air with a medication in the mist. Neno Bene is a medical device, and shall be used only with fluids and medications for mesh nebulizers. You shall not use it with oils or fluids of high density.

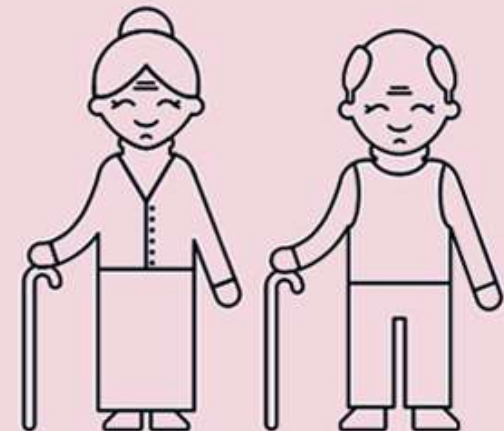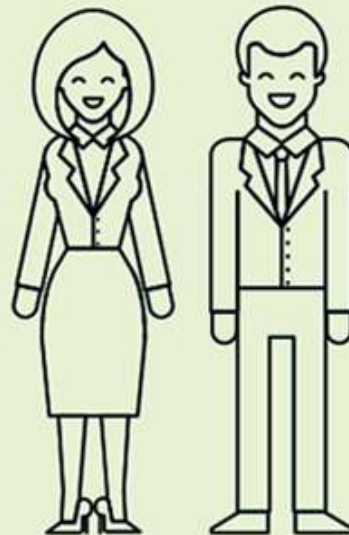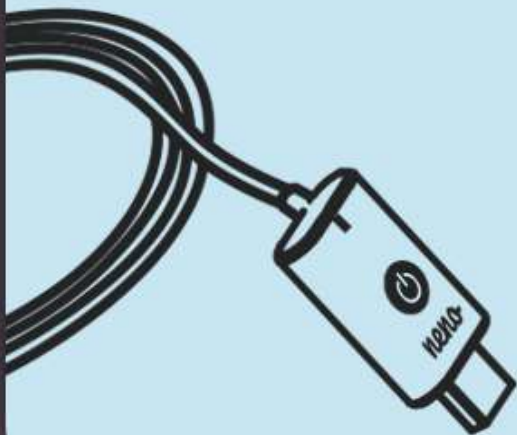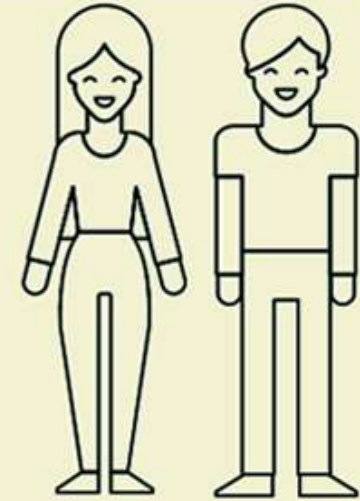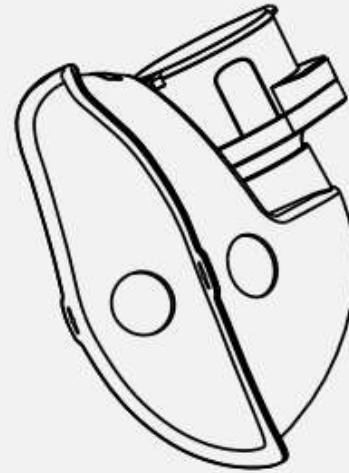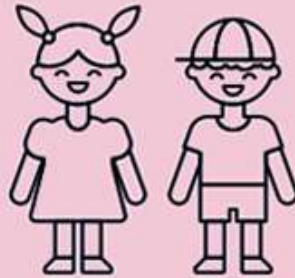

EXCEPTIONALLY QUIET OPERATING

High frequency vibrations split the fluid with medication into very tiny particles (1 – 5 microns) which can penetrate deeper into the respiratory system, and thus reach the source of infection. Frequency of vibrations is 110 thousands per second, while the volume stays below 35 dB. Therefore Neno Bene is a very quiet device, which is very helpful in many situations – at work, on a plane / train, or if your child has sensoric disorders or doesn't tolerate noisy devices.



TECHNICAL DATA

Model: Neno Bene

Charging:	DC 5V 1A
Aerosol efficiency	≥ 0.20 ml/min
Size of particles:	MMAD ok. 5µm
Container capacity:	≤ 8ml
Charging:	USB cord or AA battery x 2pcs (not included in the set)
Main unit weight:	42x34x40 mm
Whole set weight:	72 g
EAN code:	5902479672540
CN code:	9019209000
Box size:	122x122x78 mm
Box weight:	187 g
PN:	NEN-MAM-INH002

